


INFLUENZA A+B RAPID TEST CASSETTE

The Influenza A+B Rapid Test Cassette (Swab/Nasal Aspirate) is a rapid chromatographic immunoassay for the qualitative detection of Influenza A & B antigens in the nasopharyngeal swab, throat swab or nasal aspirate specimens.

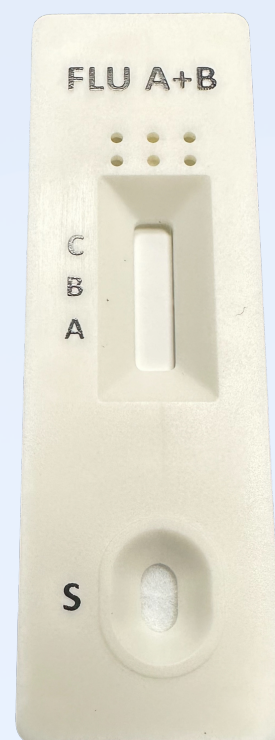

15

Results in 15 Minutes

Results are available within fifteen minutes.

2

A & B Testing

Tests against both Influenza A & B.




Easy-to-Use Nasal Swab

Five easy-to-use steps for screening results.

KIT MATERIALS

- Test Cassettes
- Sterile Swabs
- Extraction Tube Tips
- Extraction Reagent
- Extraction Tubes
- Workstation
- Package Insert

CONTACT US

 (866) 287-2425

 <https://spectrummdx.com>

INFLUENZA A+B - RAPID TEST CASSETTE

SPECIMEN COLLECTION AND PREPARATION


Nasopharyngeal swab sample

1. Insert a sterile swab into the nostril of the patient, reaching the posterior nasopharynx.
2. Swab over the surface of the posterior nasopharynx 5-10 times.

Throat swab sample

Insert a sterilized swab into pharynx and collect mucoepidermis mainly wiping flare region of post-pharyngeal wall and palatine tonsil several times, be careful not to make saliva attach to swab.

DIRECTIONS FOR USE

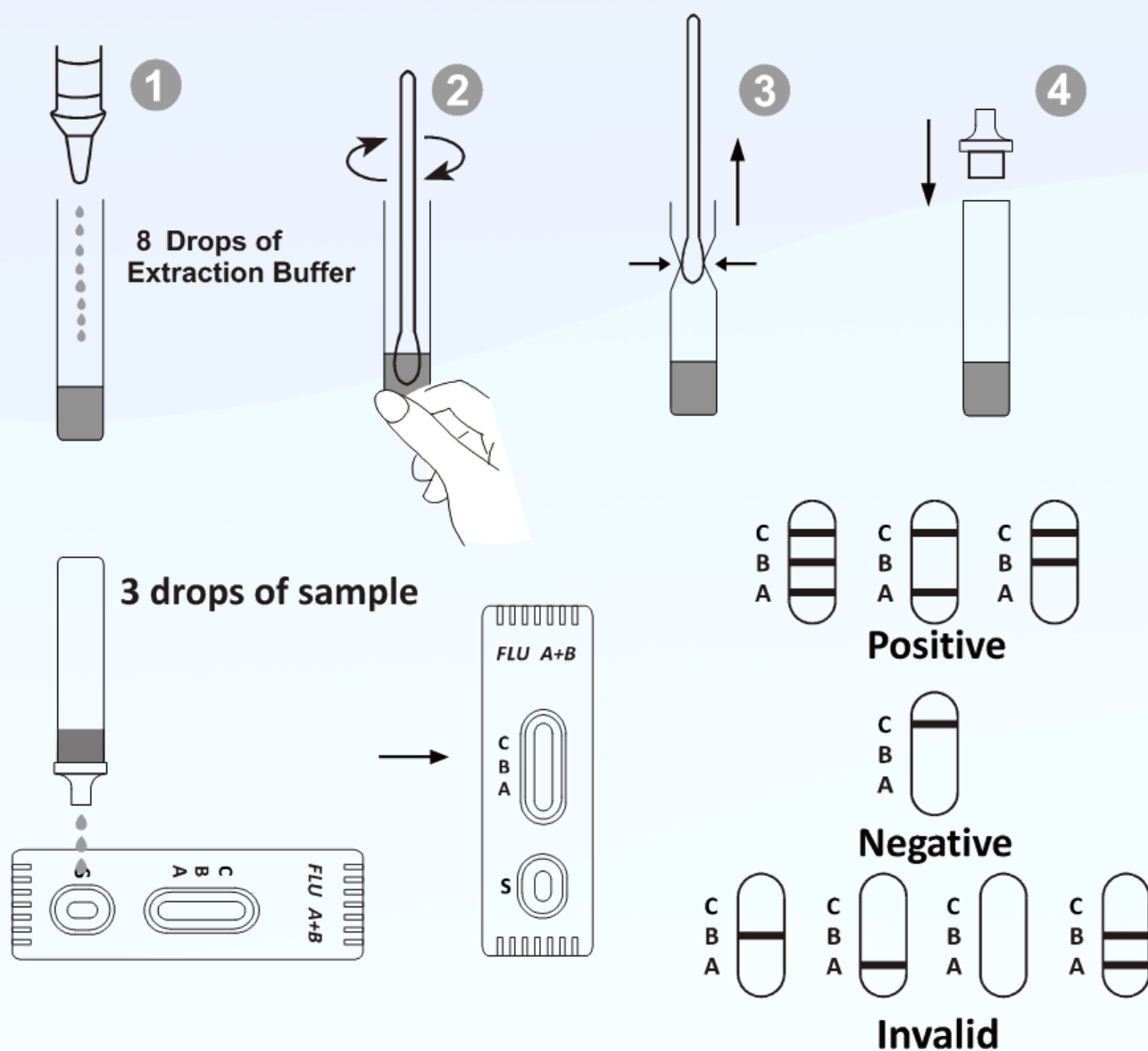

CONTACT US

(866) 287-2425

<https://spectrummdx.com>